

Introduction

Thank you for having chosen a Univox[®] product. We hope you will be satisfied!

Please read this user guide carefully before installation and use of this product.

A digital to analog converter is a product that converts digital audio signals into analog audio. It can be used if you have a loop amplifier, wireless headset or neckloop that does not have a digital input.

First, check that your audio equipment works properly and then turn it off before installation.

The package contains

- HDA-2M digital to analog converter (1.)
- Optical cable (2.)
- Power adaptor (3.)

In some product packages there is also an audio cable included.



(1.)




(2.)



(3.)

Overview

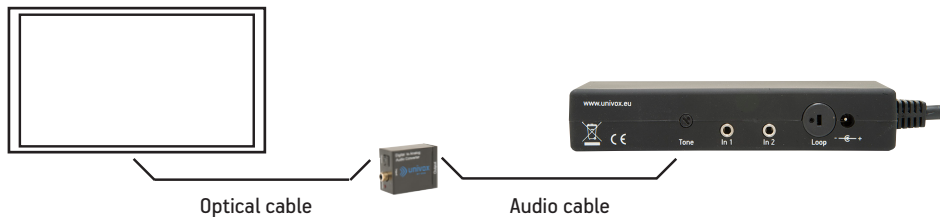


Connect the converter to the TV

1. Remove the plastic cover from the connectors on the optical cable.




2. Connect to Toslink input on the connector and to the TV. Use the Toslink/Optical Output on the TV.

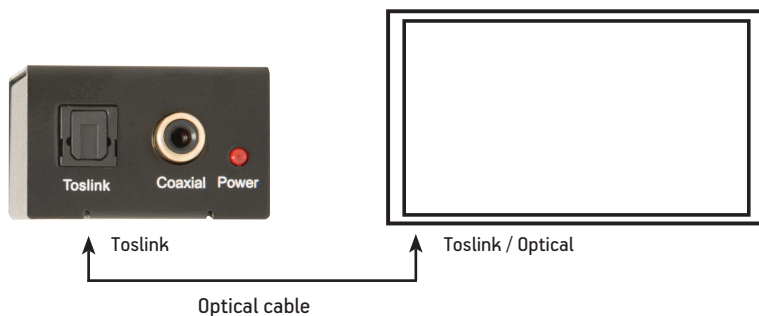

Connect the converter to the loop amplifier (or the transmitter of the wireless headset/neckloop)

Use the audio cable that was delivered with the loop amplifier/transmitter or was included with the connector.

1. Connect the red and white connector to the converter (R/L)
2. Connect to the input (3.5 mm) to the amplifier/transmitter. The input is marked IN or AUDIO IN.

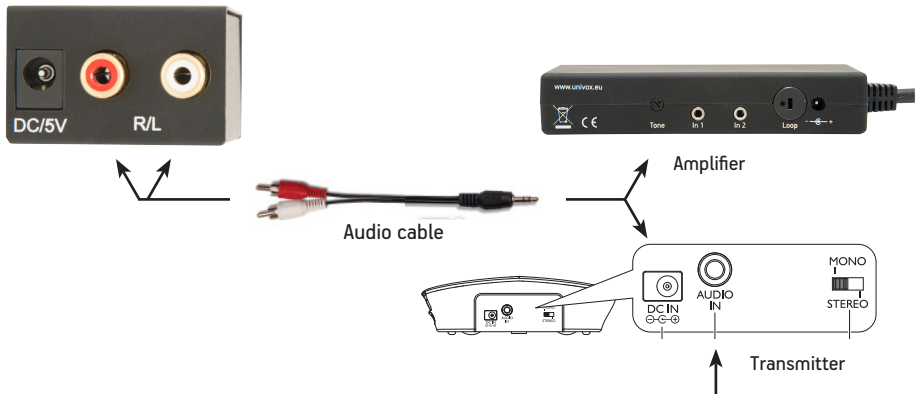

Connect the converter to the mains power

1. Connect the power adaptor to the DC/5V on the connector and to the mains power.
2. Connect the power adaptor of the loop amplifier/transmitter to the mains power.



Security/warranty

Please note that the warranty is not valid for any damage or defects on the product due to incorrect or incautious installation or usage or maintenance.

Do not use any other power adaptor than the one supplied with this product.

Bo Edin AB is not responsible for interference from radio or TV equipment, or damage caused by the product to any property or liable for the consequences of such damages.

Maintenance and care

Under normal circumstances the product does not need any special maintenance. Handle the product with care to extend its function and performance.

Do not install the product where it is exposed to strong sunlight, radiators or other heat sources, rain, water or moist environments.

Should the unit become dirty, wipe it with a clean damp (not wet!) cloth. Do not use solvent or strong detergents.

Troubleshooting

Make sure all connections are connected according to this user guide. You can try to remove them and connect them again.

Service

Should the product not work after having made the test as described above, please contact the local distributor of the product for further instructions. If the product should be sent to Bo Edin AB, please enclose a filled Service Form (pdf), www.univox.eu/documents.

Technical data



For additional information, please refer to product data sheet/brochure and CE certificate which can be downloaded from www.univox.eu/products. If required other technical documents can be ordered from support@edin.se.

Environment



When this product is finished with, please follow existing disposal regulations. Thus if you respect these instructions you ensure human health and environmental protection.

Distributor

Univox by edin, the world's leading expert and producer of high quality hearing loop systems, created the very first true loop amplifier 1969. Ever since our mission is to serve the hearing community with the highest degree of service and performance with strong focus on Research and Development for new technical solutions.

Bo Edin AB, Stockby Hantverksby 3, SE-181 75 Lidingö, Sweden | www.univox.eu

Hearing excellence since 1965

